

TPLO Plate System


Surgical Technique for Tibial Plateau Leveling Osteotomy
With *InternalBrace*™ Ligament Augmentation



Arthrex®
Vet Systems

Locking TPLO plate
with *Internal/Brace™* ligament
augmentation option

Laser line assists in
proper placement
across the osteotomy


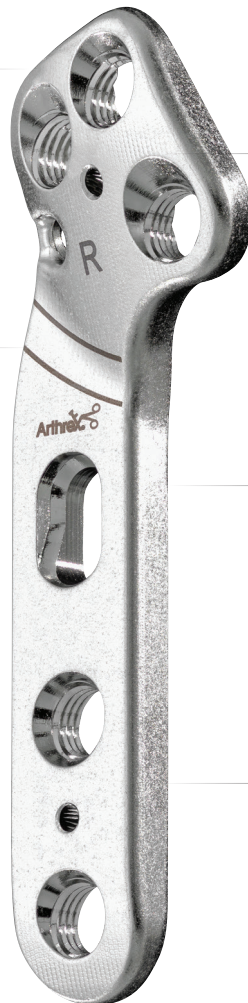


New shape and additional
features designed to facilitate
optimal plate positioning


Designed to be less
prominent under the skin

Locking screws
proximally and distally


Tibial Plateau Leveling Osteotomy With *Internal/Brace*™ Ligament Augmentation

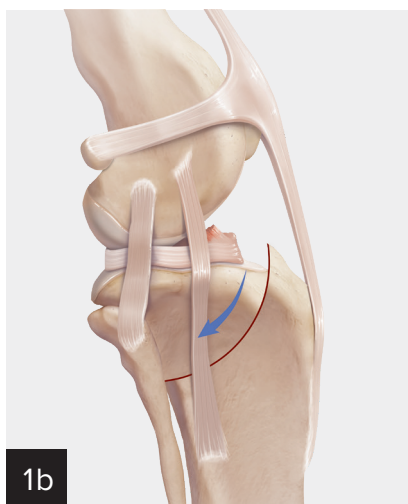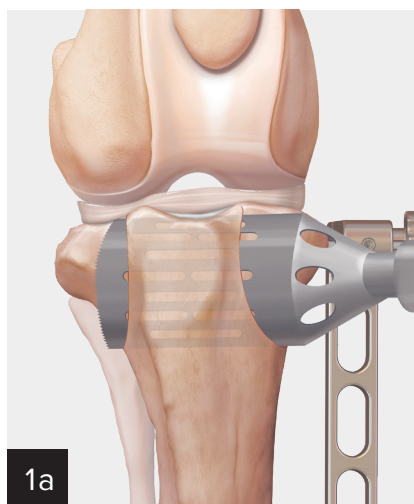


Perform a tibial plateau leveling osteotomy (TPLO) with or without a standard TPLO jig and the appropriate size saw blade. Secure the osteotomy using a Kirschner wire (K-wire) placed across the tibial tuberosity.

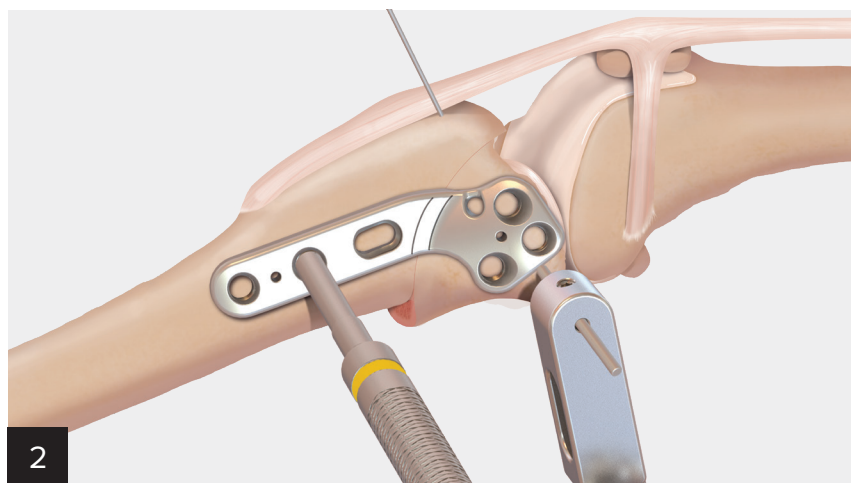


To properly align the TPLO plate, place the proximal screw holes in the center of the proximal fragment. Place the distal portion of the plate in the center of the distal fragment and align the laser lines with the osteotomy.

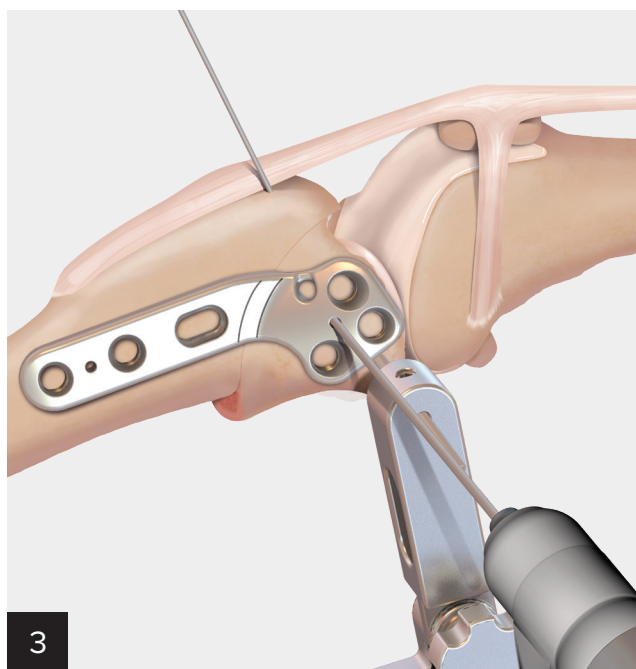

After placing the plate in the center of the proximal fragment, insert the proximal K-wire.

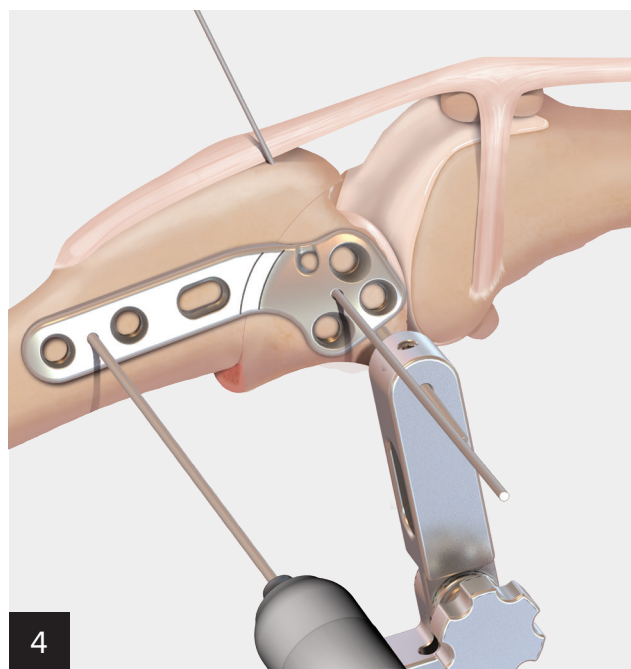


After aligning the plate with the tibial diaphysis, insert the distal K-wire to secure the plate.

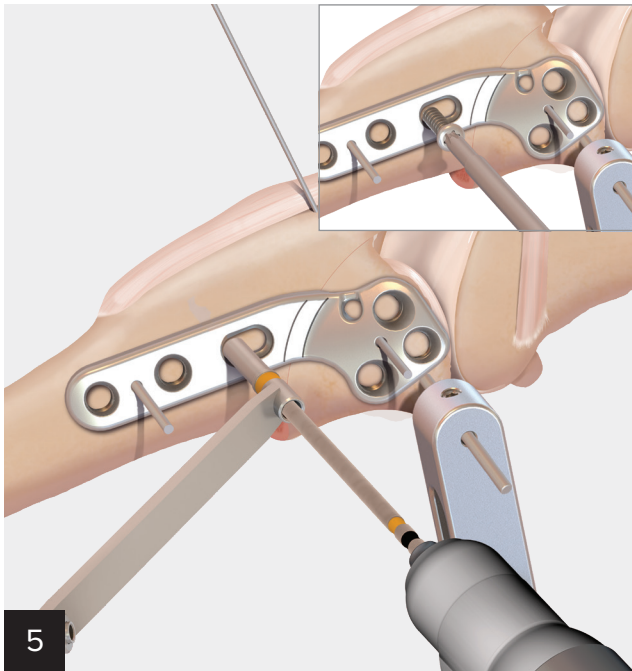


5

Insert a cortical screw in compression position. This screw should be left slightly loose but in contact with the plate. Use the drill guide and appropriate drill bit to drill a hole in the compression position.

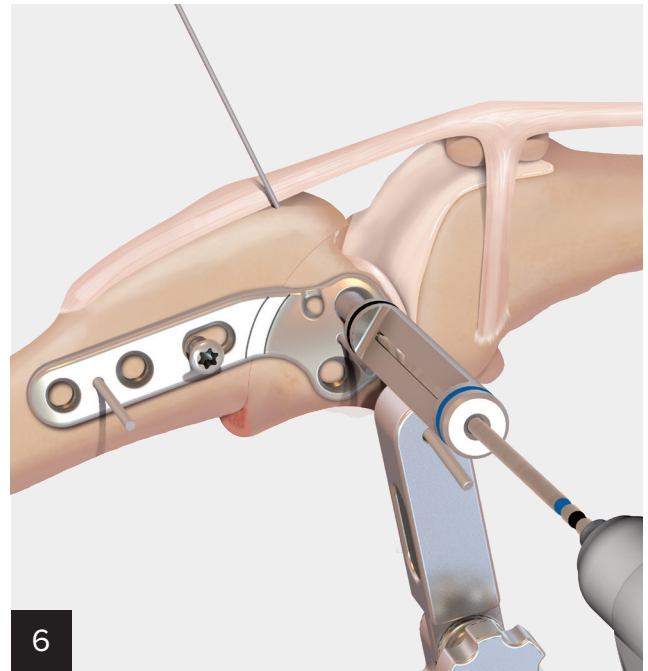


6

Use the locking drill guide to drill the hole and to measure its depth to the appropriate length. Use a depth gauge for a more accurate measurement.

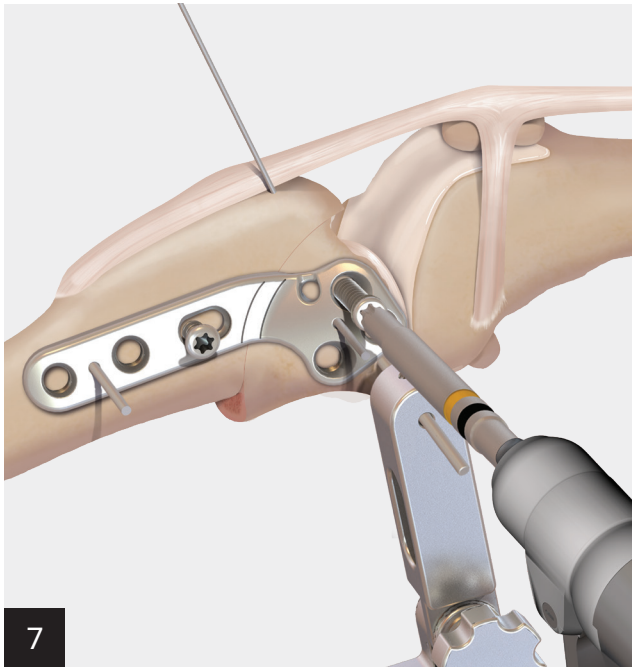


7

Place the locking screws in the proximal fragment.

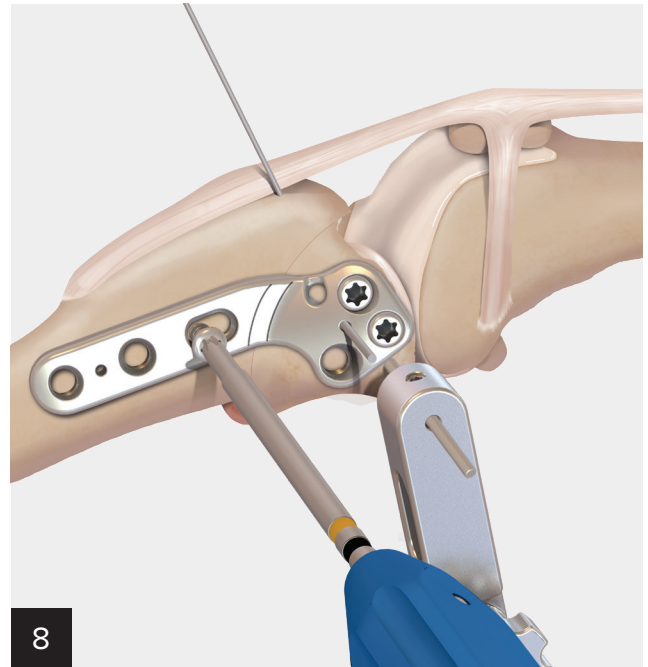


8


After removing the distal K-wire, tighten the compression screw by hand.

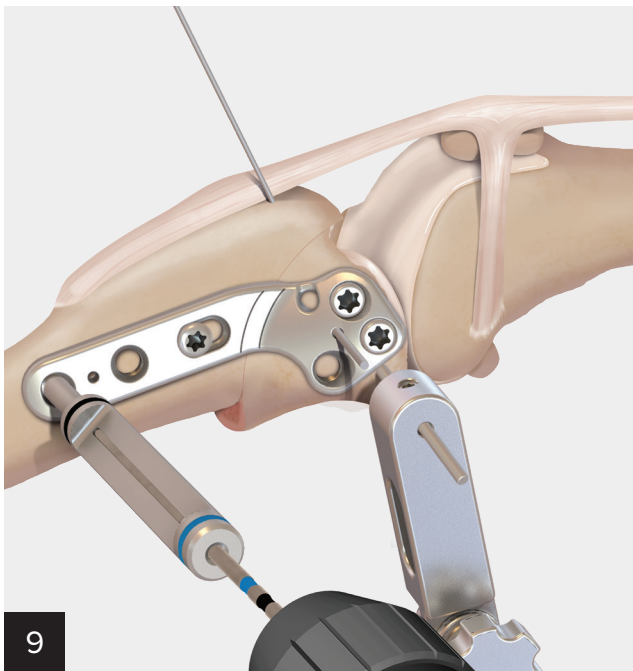


9
After tightening the compression cortical screw, insert the locking screws in the distal holes and remove the jig.

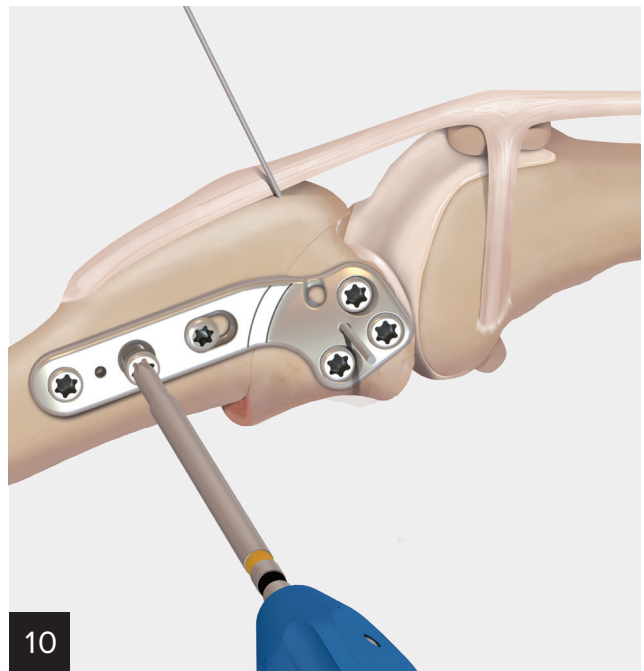


10
After all screws are placed, use a screwdriver to tighten each screw by hand and then remove the K-wire.

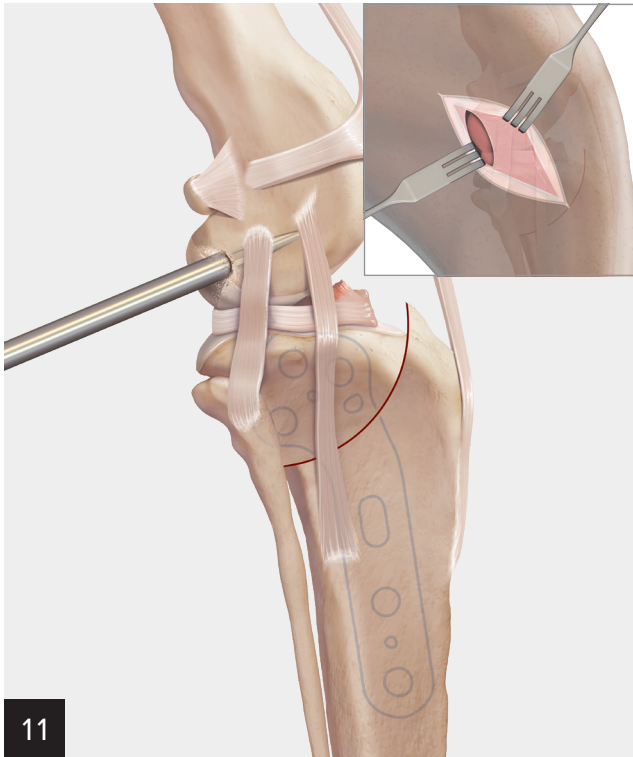


11
Expose the caudal aspect of the lateral femoral condyle. Drill holes for the SwiveLock® anchor using the appropriate size bit.

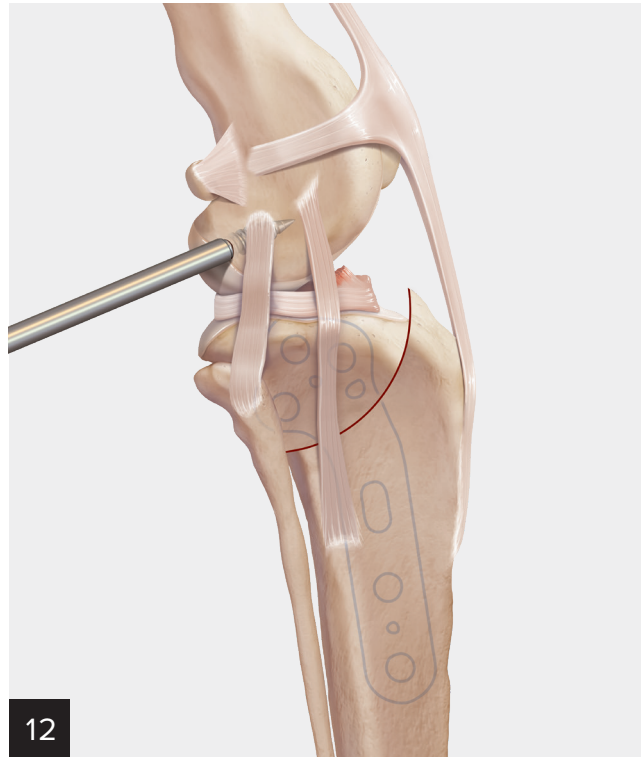


12
Tap the lateral femoral drill hole for the SwiveLock anchor using the appropriate size tap to create threads for the anchor to be inserted.

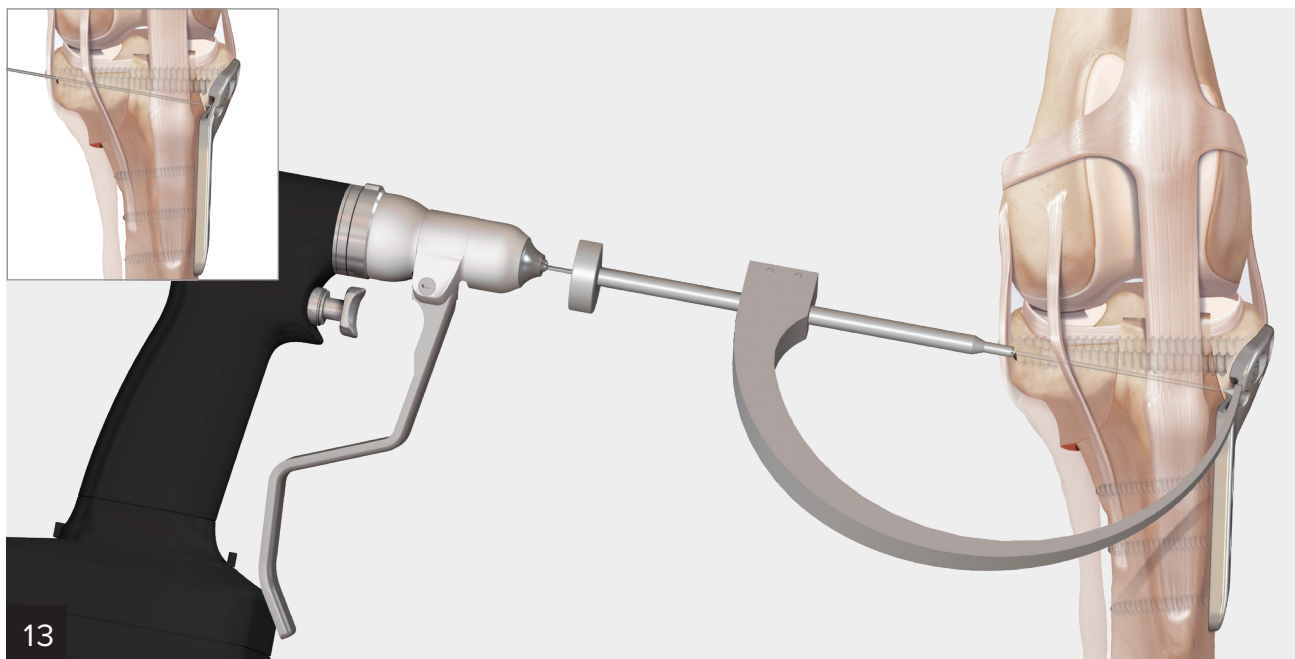


After exposing the tibial tunnel starting point behind the long digital extensor (LDE) groove, place the tip of the aiming guide in the suture hole of the plate and cinch the drill guide sleeve into place to secure the drill guide to the tibia. Drill the appropriately sized K-wire for the tibial tunnel, lateral to medial, making sure the K-wire exits near the open hole in the TPLO plate. Remove the drill guide after properly placing the K-wire.

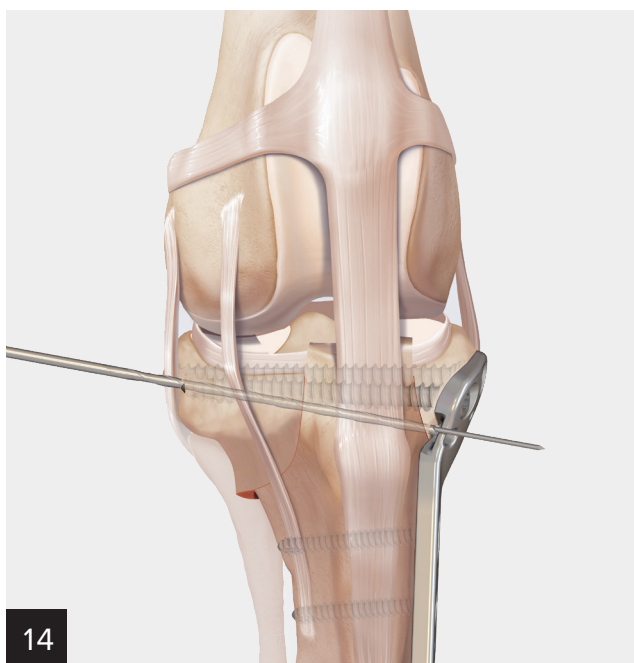


After placing the appropriately sized K-wire, use the appropriate size cannulated drill bit to create the tunnel.

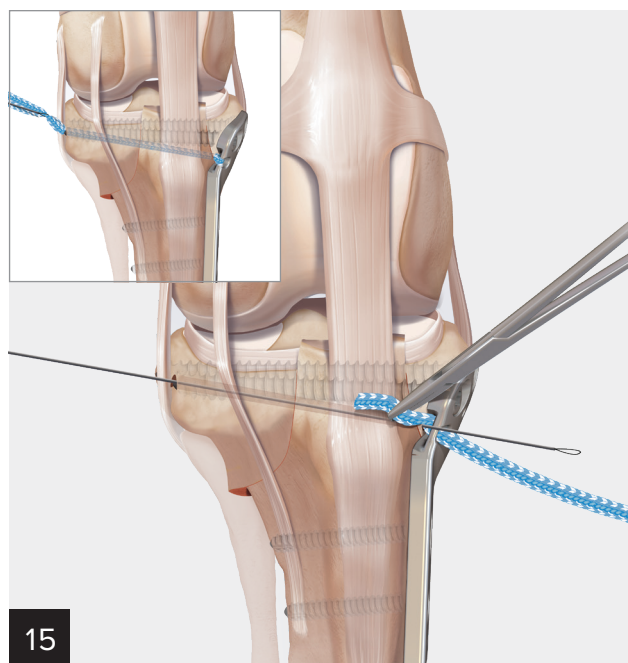


Loop the suture material through the suture hole on the TPLO plate and shuttle it through the tibial tunnel using the nitinol suture passer.

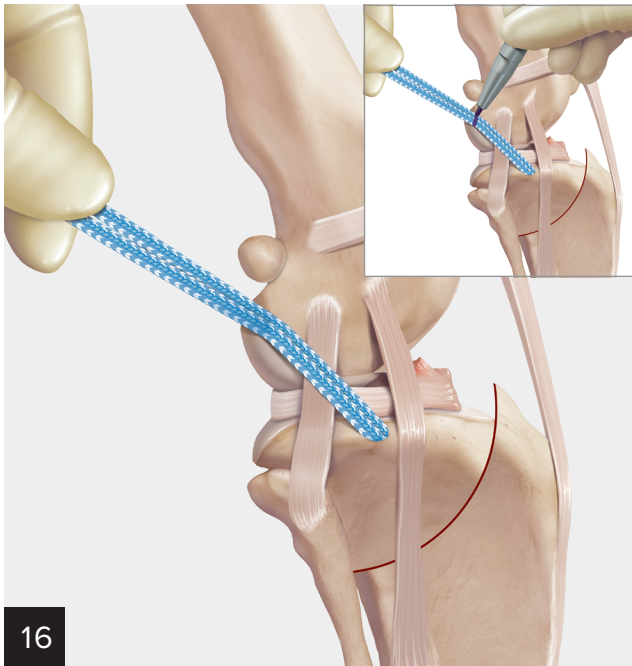


16 Pull both limbs of the suture material taut and lay over the predrilled hole in femur. Use a pen to mark the location of the center of the hole on the suture (inset).

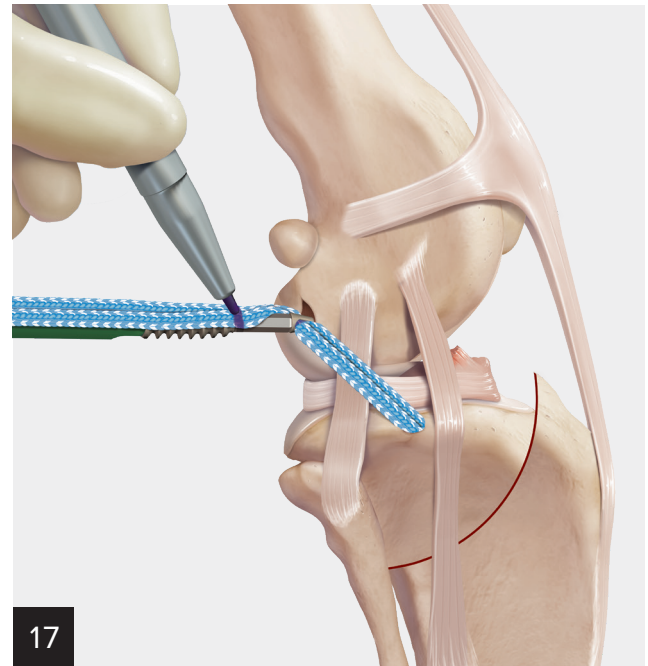


17 Slide the anchor eyelet over the mark created in Step 16. Pull both limbs of the suture material up the shaft of the SwiveLock® anchor and mark the location of the laser line on the suture.

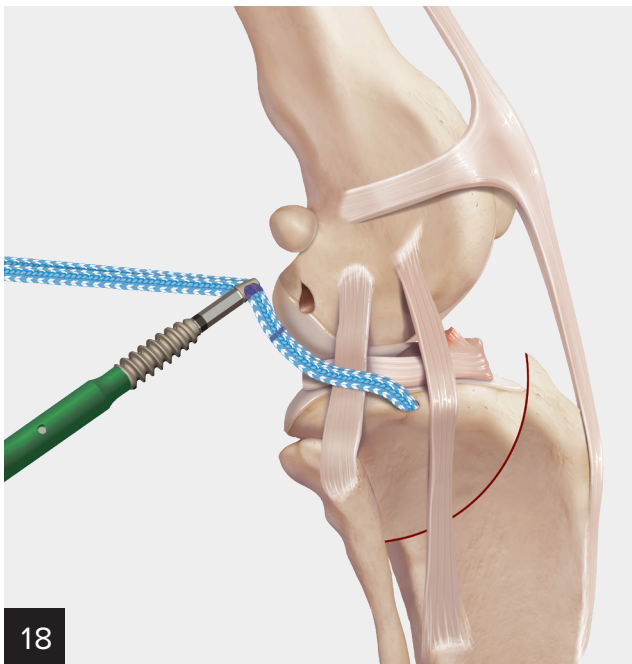

18 Move the eyelet to the mark on the suture material created in Step 17 and advance the anchor and suture into the F2 tunnel. If necessary, use a mallet to advance the eyelet into the tunnel until the tip of the anchor is at the entrance to the femoral hole.

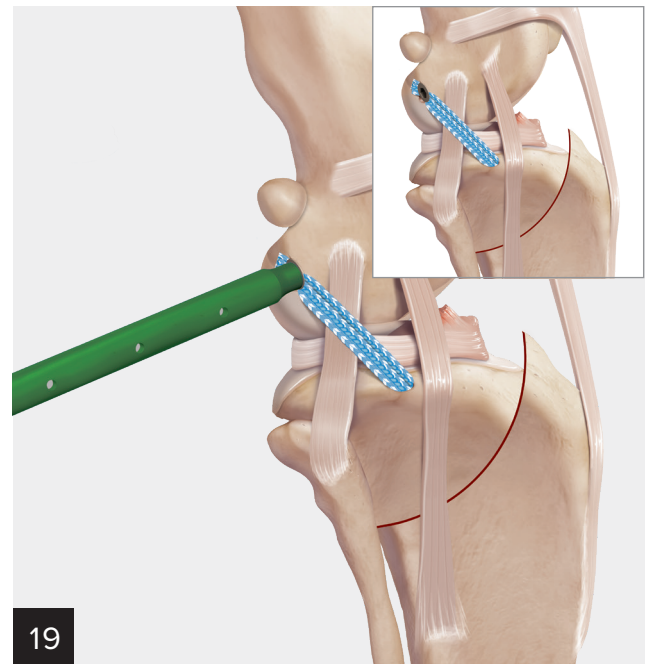


19 If the joint is secure, advance the anchor fully into the tunnel. Remove the driver from the anchor in the appropriate manner. Cut excess suture flush at the anchor.

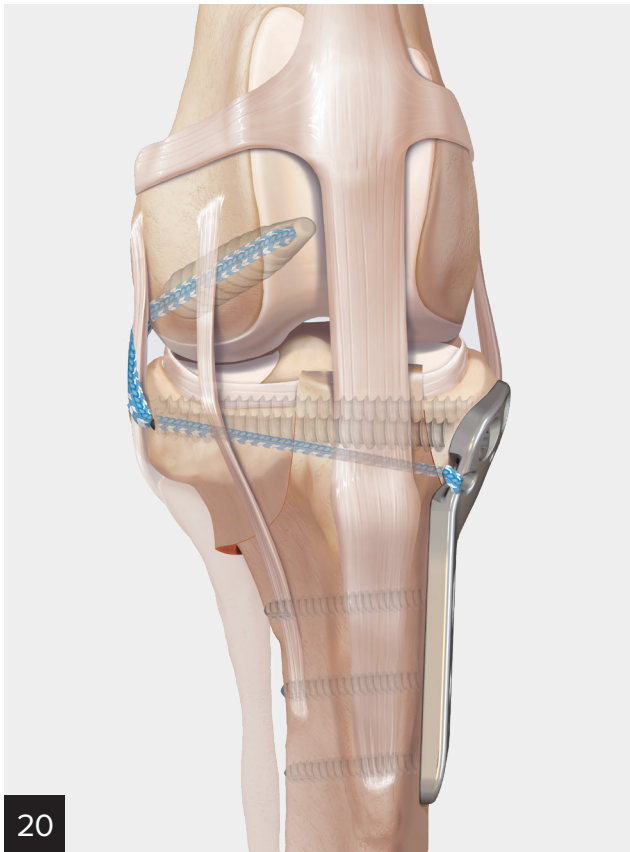

20

Cranial view.

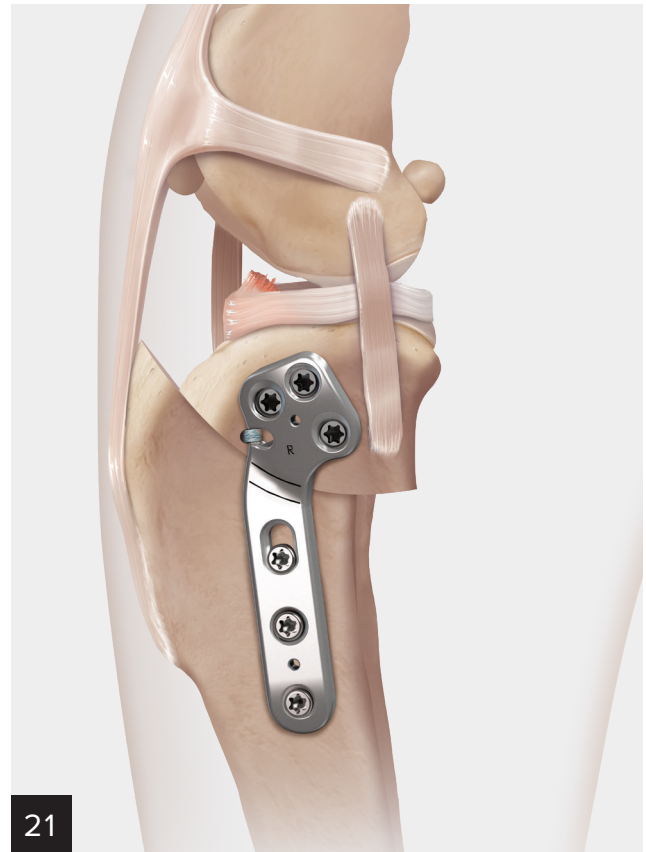


21


Medial view.

Surgical Pearls

- Preoperative planning allows repeatable and accurate placement of the osteotomy
- Difficult fragment rotation may be due to poorly centered osteotomy, incomplete osteotomy, or tibia-fibular synostosis
- The stifle should be held in flexion while sawing to prevent the popliteal vessels being compressed against the caudal aspect of the tibia (risk of bleeding)
- No attempt should be made to align the medial cortices of the tibial fragments
- Better interfragmentary compression can be achieved with the following order:
 - 2 locking screws in the proximal fragment, and
 - Compression screw in the distal fragment
- The *InternalBrace*™ augmentation should not be overtightened. The tibia should be free to rotate to neutral-mild internal rotation (about 5-10 degrees)

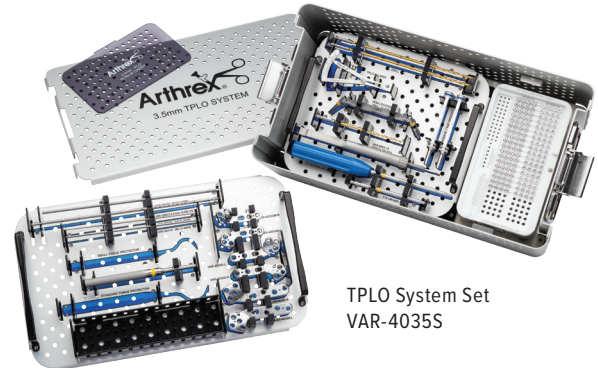

TPLO System Set
VAR-4035S

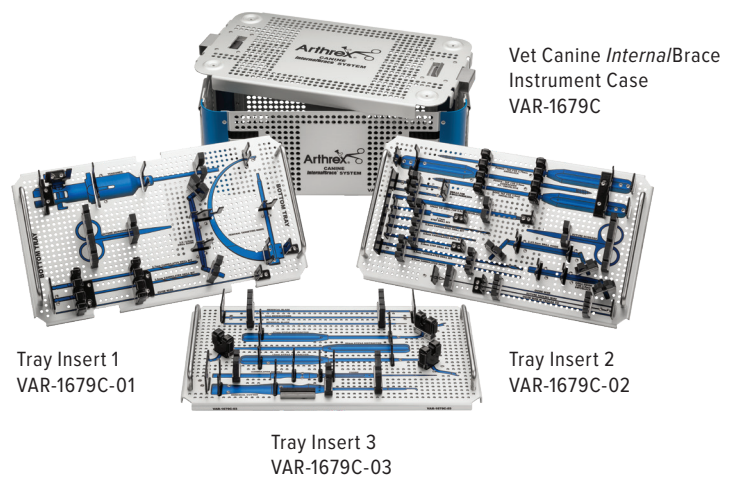

Tray Insert 1
VAR-1679C-01

Tray Insert 2
VAR-1679C-02

Tray Insert 3
VAR-1679C-03

2.7 mm TPLO Ordering Information

2.7 mm Plates

Product Description	Item Number
TPLO Locking Plate, 2.7 mm, standard, left	VAR-4027S-L
TPLO Locking Plate, 2.7 mm, standard, right	VAR-4027S-R


2.7 mm Low-Profile, Nonlocking Screws, Stainless Steel, Cortical

Product Description	Item Number
Low-Profile Cortical Screw, 2.7 mm × 10 mm	VAR-8827-10
Low-Profile Cortical Screw, 2.7 mm × 12 mm	VAR-8827-12
Low-Profile Cortical Screw, 2.7 mm × 14 mm	VAR-8827-14
Low-Profile Cortical Screw, 2.7 mm × 16 mm	VAR-8827-16
Low-Profile Cortical Screw, 2.7 mm × 18 mm	VAR-8827-18
Low-Profile Cortical Screw, 2.7 mm × 20 mm	VAR-8827-20
Low-Profile Cortical Screw, 2.7 mm × 22 mm	VAR-8827-22
Low-Profile Cortical Screw, 2.7 mm × 24 mm	VAR-8827-24
Low-Profile Cortical Screw, 2.7 mm × 26 mm	VAR-8827-26
Low-Profile Cortical Screw, 2.7 mm × 28 mm	VAR-8827-28
Low-Profile Cortical Screw, 2.7 mm × 30 mm	VAR-8827-30
Low-Profile Cortical Screw, 2.7 mm × 32 mm	VAR-8827-32
Low-Profile Cortical Screw, 2.7 mm × 34 mm	VAR-8827-34

2.7 mm Low-Profile, Locking Screws, Stainless Steel

Product Description	Item Number
Low-Profile Locking Screw, 2.7 mm × 10 mm	VAR-8827L-10
Low-Profile Locking Screw, 2.7 mm × 12 mm	VAR-8827L-12
Low-Profile Locking Screw, 2.7 mm × 14 mm	VAR-8827L-14
Low-Profile Locking Screw, 2.7 mm × 16 mm	VAR-8827L-16
Low-Profile Locking Screw, 2.7 mm × 18 mm	VAR-8827L-18
Low-Profile Locking Screw, 2.7 mm × 20 mm	VAR-8827L-20
Low-Profile Locking Screw, 2.7 mm × 22 mm	VAR-8827L-22
Low-Profile Locking Screw, 2.7 mm × 24 mm	VAR-8827L-24
Low-Profile Locking Screw, 2.7 mm × 26 mm	VAR-8827L-26
Low-Profile Locking Screw, 2.7 mm × 28 mm	VAR-8827L-28
Low-Profile Locking Screw, 2.7 mm × 30 mm	VAR-8827L-30
Low-Profile Locking Screw, 2.7 mm × 32 mm	VAR-8827L-32
Low-Profile Locking Screw, 2.7 mm × 34 mm	VAR-8827L-34

Additional screw sizes may be available; please contact your Arthrex representative for additional information.

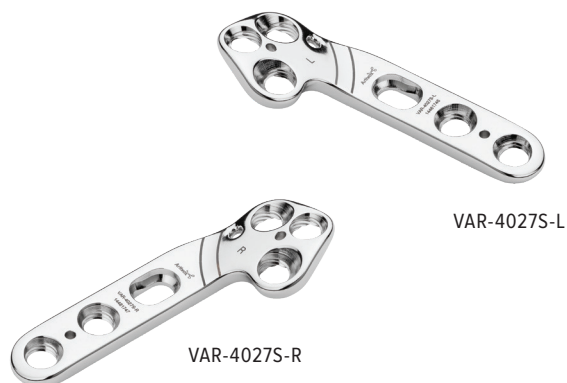

Instruments

Product Description	Item Number
TPLO Tissue Protector, small	VAR-4000TPSM
TPLO Caliper	VAR-4000-CAL
Nonlocking Drill Guide, 2.0 mm/3.0 mm	VAR-8943-31
Locking Drill Guide, 2.7 mm	VAR-8950-07
Depth Device, low profile, 2.7 mm/3.5 mm	VAR-8943-15
T10 Driver	VAR-8944DH
T10 Screwdriver	VAR-8943-08
Screw Holding Forceps	VAR-8941F
Locking Plate Holder, 2.7 mm	VAR-8950-09
Mini Mallet, 7 in	VAR-8826M
Low-Profile Hohmann Retractor	VAR-13210
Wire Cutter, 0.062 in	VAR-8956-10

Disposables and Limited Reusables

Product Description	Item Number
K-Wire, 1.14 mm (0.045 in)	VAR-8933K
K-Wire, 1.57 mm (0.062 in)	VAR-8941K
K-Wire, 2.3 mm (0.092 in)	VAR-8967K
Drill Bit, 2.0 mm	VAR-8944-22
Small Osteotome, 5 mm	VAR-13203-05

Sets/Cases

Product Description	Item Number
TPLO Set With Instruments, 2.7 mm	VAR-4027S
TPLO Case With Screw Caddy, 2.7 mm	VAR-4027C
TPLO Screw Caddy, 2.7 mm	VAR-4027SC-01

InternalBrace™ Ligament Augmentation System

Product Description	Item Number
Implants for InternalBrace System	
SwiveLock® Anchor, 3.5 mm	VAR-2325PSLC
FiberTape® Suture, 2.0 mm	AR-7237
LabralTape™ Suture, 1.5 mm	AR-7276
Instrumentation for InternalBrace System	
SwiveLock Drill Guide, 3.5 mm	VAR-1678-01
SwiveLock Tap, 3.5 mm	VAR-1678-03
SwiveLock Drill Bit, 3.5 mm	VAR-1678-05
Aiming Guide	VAR-2810
Disposables/Limited Reusables for InternalBrace System	
K-Wire, 1.14 mm (0.045 in)	VAR-8933K
K-Wire, 1.24 mm (0.049 in)	VAR-8920P
Suture Passing Wire	VAR-1255-08
Cannulated Drill Bit, 2.0 mm	VAR-8933-20C
Cannulated Drill Bit, 2.5 mm	VAR-8737-09

3.5 mm TPLO Ordering Information

3.5 mm Plates

Product Description	Item Number
TPLO Locking Plate, 3.5 mm, small, left	VAR-4035SM-L
TPLO Locking Plate, 3.5 mm, small, right	VAR-4035SM-R
TPLO Locking Plate, 3.5 mm, standard, left	VAR-4035S-L
TPLO Locking Plate, 3.5 mm, standard, right	VAR-4035S-R
TPLO Locking Plate, 3.5 mm, broad, left	VAR-4035B-L
TPLO Locking Plate, 3.5 mm, broad, right	VAR-4035B-R

3.5 mm Low-Profile, Nonlocking Screws, Stainless Steel, Cortical

Product Description	Item Number
Low-Profile Cortical Screw, 3.5 mm × 16 mm	VAR-8835-16
Low-Profile Cortical Screw, 3.5 mm × 18 mm	VAR-8835-18
Low-Profile Cortical Screw, 3.5 mm × 20 mm	VAR-8835-20
Low-Profile Cortical Screw, 3.5 mm × 22 mm	VAR-8835-22
Low-Profile Cortical Screw, 3.5 mm × 24 mm	VAR-8835-24
Low-Profile Cortical Screw, 3.5 mm × 26 mm	VAR-8835-26
Low-Profile Cortical Screw, 3.5 mm × 28 mm	VAR-8835-28
Low-Profile Cortical Screw, 3.5 mm × 30 mm	VAR-8835-30
Low-Profile Cortical Screw, 3.5 mm × 32 mm	VAR-8835-32
Low-Profile Cortical Screw, 3.5 mm × 34 mm	VAR-8835-34
Low-Profile Cortical Screw, 3.5 mm × 36 mm	VAR-8835-36
Low-Profile Cortical Screw, 3.5 mm × 38 mm	VAR-8835-38
Low-Profile Cortical Screw, 3.5 mm × 40 mm	VAR-8835-40
Low-Profile Cortical Screw, 3.5 mm × 42 mm	VAR-8835-42
Low-Profile Cortical Screw, 3.5 mm × 44 mm	VAR-8835-44
Low-Profile Cortical Screw, 3.5 mm × 46 mm	VAR-8835-46
Low-Profile Cortical Screw, 3.5 mm × 48 mm	VAR-8835-48
Low-Profile Cortical Screw, 3.5 mm × 50 mm	VAR-8835-50

3.5 mm Low-Profile, Locking Screws, Stainless Steel

Product Description	Item Number
Low-Profile Locking Screw, 3.5 mm × 16 mm	VAR-8835L-16
Low-Profile Locking Screw, 3.5 mm × 18 mm	VAR-8835L-18
Low-Profile Locking Screw, 3.5 mm × 20 mm	VAR-8835L-20
Low-Profile Locking Screw, 3.5 mm × 22 mm	VAR-8835L-22
Low-Profile Locking Screw, 3.5 mm × 24 mm	VAR-8835L-24
Low-Profile Locking Screw, 3.5 mm × 26 mm	VAR-8835L-26
Low-Profile Locking Screw, 3.5 mm × 28 mm	VAR-8835L-28
Low-Profile Locking Screw, 3.5 mm × 30 mm	VAR-8835L-30
Low-Profile Locking Screw, 3.5 mm × 32 mm	VAR-8835L-32
Low-Profile Locking Screw, 3.5 mm × 34 mm	VAR-8835L-34
Low-Profile Locking Screw, 3.5 mm × 36 mm	VAR-8835L-36
Low-Profile Locking Screw, 3.5 mm × 38 mm	VAR-8835L-38
Low-Profile Locking Screw, 3.5 mm × 40 mm	VAR-8835L-40
Low-Profile Locking Screw, 3.5 mm × 42 mm	VAR-8835L-42
Low-Profile Locking Screw, 3.5 mm × 44 mm	VAR-8835L-44
Low-Profile Locking Screw, 3.5 mm × 46 mm	VAR-8835L-46
Low-Profile Locking Screw, 3.5 mm × 48 mm	VAR-8835L-48
Low-Profile Locking Screw, 3.5 mm × 50 mm	VAR-8835L-50



Instruments

Product Description	Item Number
TPLO Tissue Protector, small	VAR-4000TPSM
TPLO Tissue Protector, standard	VAR-4000TPS
TPLO Caliper	VAR-4000-CAL
Nonlocking Drill Guide, 2.5 mm/3.5 mm	VAR-8943-14
Locking Drill Guide, 3.5 mm	VAR-4035DG
Locking Drill Guide, 3.5 mm, cortical screw	VAR-8943-43
Depth Device, low profile, 2.7 mm/3.5 mm	VAR-8943-15
T15 Driver	VAR-8941DH
T15 Screwdriver	VAR-8943-10
Screw Holding Forceps	VAR-8941F
Locking Plate Holder, 3.5 mm	VAR-8954-07
Mini Mallet, 7 in	VAR-8826M
Low-Profile Hohmann Retractor	VAR-13210
Wire Cutter, 1.57 mm (0.062 in)	VAR-8956-10

Disposables and Limited Reusables

Product Description	Item Number
K-Wire, 1.14 mm (0.045 in)	VAR-8933K
K-Wire, 1.57 mm (0.062 in)	VAR-8941K
K-Wire, 2.3 mm (0.092 in)	VAR-8967K
Guide Pin, 3.0 mm (0.118 in)	VAR-13303-3.0
Drill Bit, 2.8 mm, long	VAR-4035D
Drill Bit, 2.5 mm, short	VAR-8943-30
Drill Bit, 2.5 mm, long	VAR-8943-42
Small Osteotome, 5 mm	VAR-13203-05

Sets/Cases

Product Description	Item Number
TPLO Set With Instruments, 3.5 mm	VAR-4035S
TPLO Case With Screw Caddy, 3.5 mm	VAR-4035C
TPLO Screw Caddy, 3.5 mm	VAR-4035SC-01

InternalBrace™ Ligament Augmentation System

Product Description	Item Number
Implants for InternalBrace System	
SwiveLock® Anchor, 4.75 mm	VAR-2324PSLC
FiberTape® Suture, 2.0 mm	AR-7237
Instrumentation for InternalBrace System	
Spade Tip Drill Bit, 4.75 mm/5.5 mm SwiveLock suture	VAR-1927D
SwiveLock Tap, 4.75 mm	VAR-2324PTB
Aiming Guide	VAR-2810
Disposables/Limited Reusables for InternalBrace System	
K-Wire, 1.14 mm (0.045 in)	VAR-8933K
K-Wire, 1.24 mm (0.049 in)	VAR-8920P
Suture Passing Wire	VAR-1255-08
Cannulated Drill Bit, 2.5 mm	VAR-8737-09
Cannulated Drill Bit, 2.7 mm	VAR-8911DC

4.5 mm TPLO Ordering Information

4.5 mm Plates


Product Description	Item Number
TPLO Locking Plate, 4.5 mm, standard, left	VAR-4045S-L
TPLO Locking Plate, 4.5 mm, standard, right	VAR-4045S-R

4.5 mm Low-Profile, Nonlocking Screws, Stainless Steel, Cortical

Product Description	Item Number
Low-Profile Cortical Screw, 4.5 mm × 24 mm	VAR-8845-24
Low-Profile Cortical Screw, 4.5 mm × 26 mm	VAR-8845-26
Low-Profile Cortical Screw, 4.5 mm × 28 mm	VAR-8845-28
Low-Profile Cortical Screw, 4.5 mm × 30 mm	VAR-8845-30
Low-Profile Cortical Screw, 4.5 mm × 32 mm	VAR-8845-32
Low-Profile Cortical Screw, 4.5 mm × 34 mm	VAR-8845-34
Low-Profile Cortical Screw, 4.5 mm × 36 mm	VAR-8845-36
Low-Profile Cortical Screw, 4.5 mm × 38 mm	VAR-8845-38
Low-Profile Cortical Screw, 4.5 mm × 40 mm	VAR-8845-40
Low-Profile Cortical Screw, 4.5 mm × 42 mm	VAR-8845-42
Low-Profile Cortical Screw, 4.5 mm × 44 mm	VAR-8845-44
Low-Profile Cortical Screw, 4.5 mm × 46 mm	VAR-8845-46
Low-Profile Cortical Screw, 4.5 mm × 48 mm	VAR-8845-48
Low-Profile Cortical Screw, 4.5 mm × 50 mm	VAR-8845-50
Low-Profile Cortical Screw, 4.5 mm × 52 mm	VAR-8845-52
Low-Profile Cortical Screw, 4.5 mm × 54 mm	VAR-8845-54
Low-Profile Cortical Screw, 4.5 mm × 56 mm	VAR-8845-56
Low-Profile Cortical Screw, 4.5 mm × 58 mm	VAR-8845-58
Low-Profile Cortical Screw, 4.5 mm × 60 mm	VAR-8845-60
Low-Profile Cortical Screw, 4.5 mm × 65 mm	VAR-8845-65

4.5 mm Low-Profile, Locking Screws, Stainless Steel

Product Description	Item Number
Low-Profile Locking Screw, 4.5 mm × 24 mm	VAR-8845L-24
Low-Profile Locking Screw, 4.5 mm × 26 mm	VAR-8845L-26
Low-Profile Locking Screw, 4.5 mm × 28 mm	VAR-8845L-28
Low-Profile Locking Screw, 4.5 mm × 30 mm	VAR-8845L-30
Low-Profile Locking Screw, 4.5 mm × 32 mm	VAR-8845L-32
Low-Profile Locking Screw, 4.5 mm × 34 mm	VAR-8845L-34
Low-Profile Locking Screw, 4.5 mm × 36 mm	VAR-8845L-36
Low-Profile Locking Screw, 4.5 mm × 38 mm	VAR-8845L-38
Low-Profile Locking Screw, 4.5 mm × 40 mm	VAR-8845L-40
Low-Profile Locking Screw, 4.5 mm × 42 mm	VAR-8845L-42
Low-Profile Locking Screw, 4.5 mm × 44 mm	VAR-8845L-44
Low-Profile Locking Screw, 4.5 mm × 46 mm	VAR-8845L-46
Low-Profile Locking Screw, 4.5 mm × 48 mm	VAR-8845L-48
Low-Profile Locking Screw, 4.5 mm × 50 mm	VAR-8845L-50
Low-Profile Locking Screw, 4.5 mm × 52 mm	VAR-8845L-52
Low-Profile Locking Screw, 4.5 mm × 54 mm	VAR-8845L-54
Low-Profile Locking Screw, 4.5 mm × 56 mm	VAR-8845L-56
Low-Profile Locking Screw, 4.5 mm × 58 mm	VAR-8845L-58
Low-Profile Locking Screw, 4.5 mm × 60 mm	VAR-8845L-60
Low-Profile Locking Screw, 4.5 mm × 65 mm	VAR-8845L-65

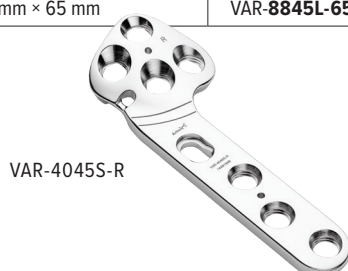

Instruments

Product Description	Item Number
TPLO Tissue Protector, standard	VAR-4000TPS
TPLO Caliper	VAR-4000-CAL
Nonlocking Drill Guide, 3.0 mm/4.5 mm	VAR-8970-02
Locking Drill Guide, 4.5 mm	VAR-4045DG
Locking Drill Guide, 4.5 mm, cortical screw	VAR-8970-01
Depth Device, 4.5 mm	VAR-8970-07
T20 Driver	VAR-8970-03
T20 Screwdriver	VAR-8970-04
T20 Screwdriver, self-retaining	VAR-4045-02
Screw Holding Forceps	VAR-8941F
Locking Plate Holder, 4.5 mm	VAR-4045-01
Mini Mallet, 7 in	VAR-8826M
Wire Cutter, 1.57 mm (0.062 in)	VAR-8956-10
Low-Profile Hohmann Retractor	VAR-13210

Disposables and Limited Reusables

Product Description	Item Number
K-Wire, 1.57 mm (0.062 in)	VAR-8941K
K-Wire, 2.3 mm (0.092 in)	VAR-8967K
Guide Pin, 3.0 mm (0.118 in)	VAR-13303-3.0
Drill Bit, 3.8 mm	VAR-4045D
Drill Bit, 3.0 mm	VAR-8970-30
Small Osteotome, 5 mm	VAR-13203-05

Sets/Cases

Product Description	Item Number
TPLO Set With Instruments, 4.5 mm	VAR-4045S
TPLO Case With Screw Caddy, 4.5 mm	VAR-4045C
TPLO Screw Caddy, 4.5 mm	VAR-4045SC-01

InternalBrace™ Ligament Augmentation System

Product Description	Item Number
Implants for InternalBrace System	
SwiveLock® Anchor, 5.5 mm	VAR-2323PSLC
FiberTape® Suture, 2.0 mm	AR-7237
Instrumentation for InternalBrace System	
Spade Tip Drill Bit, 4.75 mm/5.5 mm SwiveLock anchor	VAR-1927D
SwiveLock Tap, 5.5 mm	VAR-1927CTB
Aiming Guide	VAR-2810
Disposables/Limited Reusables for InternalBrace System	
K-Wire, 1.14 mm (0.045 in)	VAR-8933K
K-Wire, 1.24 mm (0.049 in)	VAR-8920P
Suture Passing Wire	VAR-1255-08
Cannulated Drill Bit, 2.5 mm	VAR-8737-09
Cannulated Drill Bit, 2.7 mm	VAR-8911DC

NOTES

This image shows a full page of blank, lined paper. It features approximately 20 evenly spaced horizontal grey lines across its entire width, providing a guide for handwriting or typing. The background is a clean, solid white color. There are no margins, text, or other markings present on the page.



This is not veterinary advice and Arthrex recommends that veterinarians be trained in the use of any particular product before using it in surgery. A veterinarian must always rely on his or her own professional clinical judgment when deciding whether to use a particular product. A veterinarian must always refer to the package insert, product label and / or instructions for use before using any Arthrex product. Products may not be available in all markets because product availability is subject to the regulatory and/or veterinary practices in individual markets. Please contact your Arthrex representative if you have questions about availability of products in your area.

View U.S. patent information at www.arthrex.com/corporate/virtual-patent-marking

arthrexvetsystems.com

© 2021 Arthrex, Inc. All rights reserved. VLT1-00053-EN_F